

Unident® Impre



Disinfection and cleaning of
dental impressions.



Unident® Impre is a liquid concentrate for the effective disinfection and cleaning of dental impressions, impression moulds, dentures, prosthetic devices etc.

Key product features

- Dilution : 10%
- Contact time : 15 minutes
- Suitable for alginate, silicone, polyether rubber, polysulphide and hydrocolloid impression materials
- Safe – no negative effects to impression surface or dimensions
- Excellent material compatibility
- Broad spectrum of efficacy
- CE marked – Complies with Medical Devices Directive 93/42/EEC



USF Healthcare S.A. | Rue François Perréard 4 | CH-1225 Chêne Bourg | Geneva | Switzerland
Tel + 41 22 839 79 00 | Fax + 41 22 839 79 10 | info@usfhc.com
www.usfhealthcare.com

UNIDENT
SWISS

Unident® Impre


Protocol for use

Before use

Ensure that appropriate gloves are worn.



Dilute Unident® Impre in a soaking tank at 10% (100ml per litre of water)

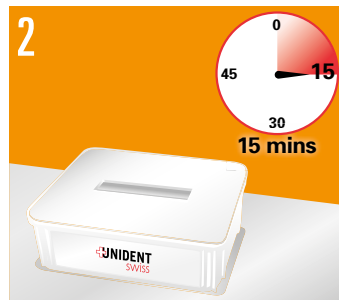

Immerse the impressions in to the solution and leave for 15 minutes.



Remove the impressions from the solution and rinse thoroughly under running water.



Dry immediately using a disposable paper towel.


Precautions

Dangerous - respect the precautions for use (established according to the Directive 99/45/CE and its adaptations).

Storage between +5°C and +35°C.

Medical Device Class IIb, CE0459 (Directive 93/42/CEE modified).

Efficacy

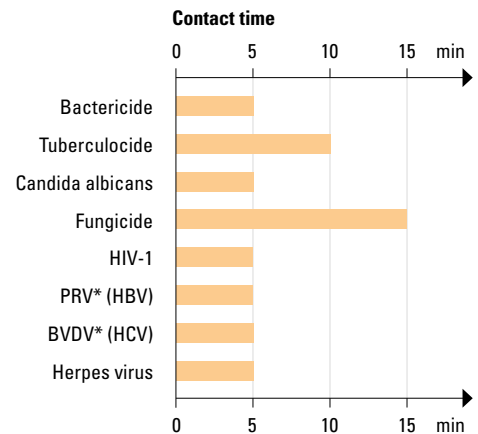

* PRV: Surrogate of Hepatitis B

* BVDV: Surrogate of Hepatitis C

Conforms to European standards

Bactericide: EN 1040, EN 13727, EN 14561
 Tuberculocide: EN 14348 (60 mins), prEN 14563
 Yeasticide: EN 1275, EN 13624
 Fungicide: EN 1275, EN 13624, EN 14562

Product composition

100g of Unident® Impre contains 2.75g N-alkyl-N-benzyl-N chloride, N-dimethylammonium and 5g glutaraldehyde.

Physical data

Colour: Colourless
 pH value: 2.5 (concentrate)
 Shelf Life: Concentrate 3 years
 Diluted 7 days max*

Container: Made of recyclable HDPE material

*Dependant on soil quantity.

Packaging

1kg	1L	2,5L	5L
	✓		

Accessories

3L Soaking tank (INTANK)



ECOEngage
Our Ecological Engagement



USF Healthcare
Unident • Steridine • Farmec

UNIDENT
SWISS