
Ethylene Oxide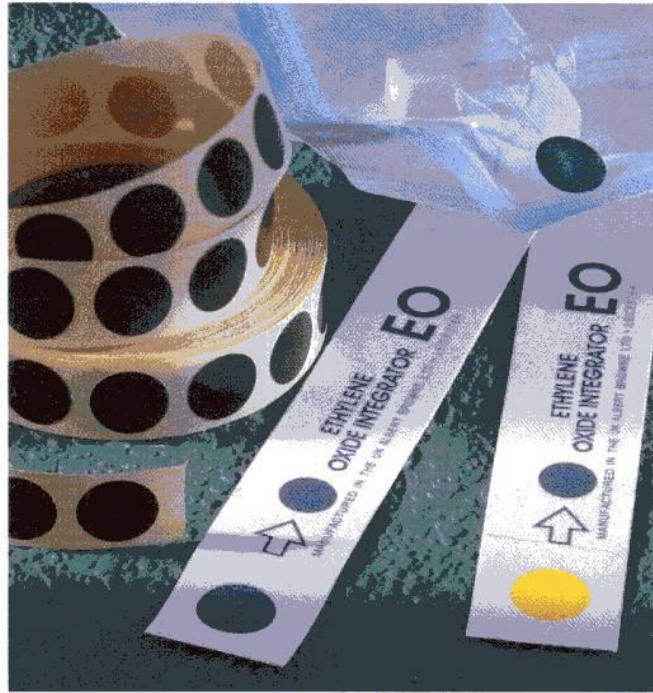

Browne Ethylene Oxide Integrators are a quick and effective way to show that EO gas has penetrated each and every pack in a sufficient quantity for the correct length of time. You can easily interpret the colour change from yellow (unprocessed) to green (pass).

One of the most difficult parameters to control in EO processes is the relative humidity level which is vital for achieving effective sterilization. Browne EO Integrators will show a fail if the relative humidity levels are outside the specified limits.

Unlike biological indicators, the Browne EO Integrators require no expensive and time consuming incubation following processing. The result can be interpreted immediately following processing allowing you to quickly and easily determine if the sterilization process has been successful.

Integrators

BEFORE AFTER

Process Detector Spots

BEFORE AFTER

Concentration (mg per litre)	Temp °C	Time	Relative Humidity
900mg	55°	20 mins	40% - 60%
	37°	55 mins	
600mg	55°	35 mins	40% - 60%
	37°	70 mins	

Chart shows some examples of the integrated criteria required to effect a colour change from yellow to green. Browne Integrators are suitable for use in most types of EO sterilizer.

These self-adhesive spots should be used on the outside of every pack being sterilized by EO gas to differentiate between processed and unprocessed packs.

Process Detector Spots
Conform to EN867-1 Class A and ISO11140-1 Class 1 [Process Indicators]

Browne Ethylene Oxide Integrators

(box/100) Re-order code: 2420

Browne Ethylene Oxide Process Detector Spots

(box/1000) Re-order code: 2421

Note: These indicator spots are not a control and only show that the packs have been through an EO process. Detector Spots change from brown to green after passing through an EO process, as shown.

Sole Exporters:
Albert Browne International Ltd.
Oakendene Estate, Bolney Road
Cowfold, Sussex RH13 8AZ England
Telephone: +44 1403 864534
Facsimile: +44 1403 864864
E-mail: brownne.int@compuserve.com

Manufactured in the UK by:
Albert Browne Ltd.
UK Enquiries:
Browne Health Care Ltd.
Chancery House, 190 Waterside Rd.
Hamilton Industrial Park,
Leicester, LE5 1QZ, UK

Tel: +44 (0)116 276 8636
Fax: +44 (0)116 276 8639

Promoting Confidence by Assuring Sterilization

Distributed by

The Company whose policy is to ensure that its equipment, reserves the right to make, without prior notification, changes in materials, dimensions and designs which, having regard to all the circumstances, it thinks reasonable or desirable provided that the customer shall not be required to accept goods substantially differing from those which are specified in the order. Illustrations, photographs and descriptions are intended as a general guide only, and shall not form part of any contract or order unless otherwise agreed to in writing. Colours shown are approximate but are as representative as printing will allow.

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